
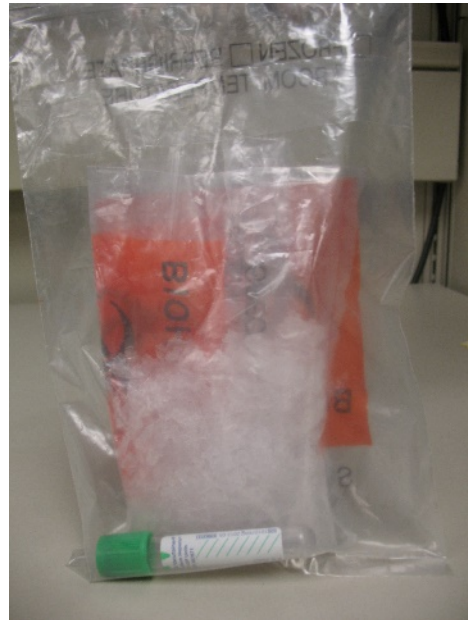


Complete the following BEFORE placing specimens on ice:

- Label on the specimen with 2 unique identifiers displayed
- Time and date the specimen was collected
- Initials of the person that obtained the specimen

Required supplies:

- 2 biohazard bags
- Ice (TIP: if you see the term “wet” ice this is regular ice as opposed to dry ice)

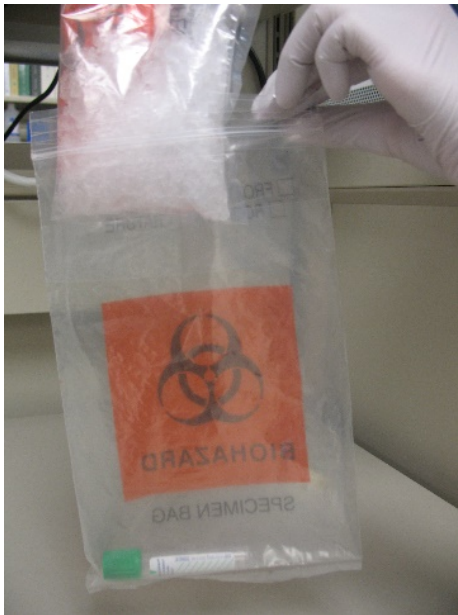

1. Place sample in 1st biohazard bag
2. Place ice in separate sealed biohazard bag
3. Place bag of ice inside 1st biohazard bag and seal

It is also acceptable to place the ice inside the biohazard bag and the sample in the outer pocket of the biohazard bag if there is a flap to cover the opening.

